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Product information

The simple and effective support for physical therapy, muscle increase and fitness

- high quality offers a wide range of applications:
- use in therapy, rehabilitation, for toning, posture correction, balance training, body awareness, improvement of flexibility
- available in 4 resistances, 2 widths and 6 lengths for progressive resistance exercises and individually adjusted applications, also with clip for safe, flexible length adjustment
- natural product made of pure rubber with reliable linear resistance behavior
- elaborate manufacturing process reduces cumbersome powdering

Safety instructions - please note

- When using Fitbands, always follow the recommendations of your doctor or therapist.
- Children may use the band only under adult supervision and after a therapy-accompanying introduction. Not suitable for infants.
- Check the band for damages (holes, cracks, cuts) prior to every use. Do not use damaged bands, replace them instead.
- Be careful with exercises at head height! Keep away from eyes, the band could slip out of hand or holder and snap in the face!
- Ensure the band cannot release from door frame or loosen from the chair or other objects.
- Ensure that door is shut while exercising.
- For bands with clip, also check the clip for damages. Make sure that the closure is locked securely in place, cannot pop open and the band cannot come loose!

Storing/Care

The SISSEL® Fitband (due to its special manufacturing process) requires only minimal powdering. If dirty, wash the band with clear water, air dry and lightly powder only when necessary. Do not expose to direct sunlight, keep away from sharp objects. Protect from damages caused by jewelry or sharp fingernails. Wash thoroughly after use in chlorinated water.

Note

Latex products can cause allergic reactions. Although the allergy-causing allergens in the SISSEL® Fitband have been reduced through a special manufacturing process, there is always the risk of an allergic reaction for the user.

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